

**BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY  
TO BE USED BY DENTIST ONLY**

**INTENDED USE**

MTA + dosing block is designated for forming precise doses of MTA+ product for the tooth cavity or root canal.



**INSTRUCTIONS FOR USE**

Prepare MTA+ product as per package leaflet supplied. On each side of the dosing block there are grooves of different sized diameters ranging from 0,7 mm to 1,00 mm.

Place the product on one side of the dosing block so that the product fills the chosen groove.

With the material formed and the desired dose selected, should be taken from the dosing block by means of a sterile instrument.



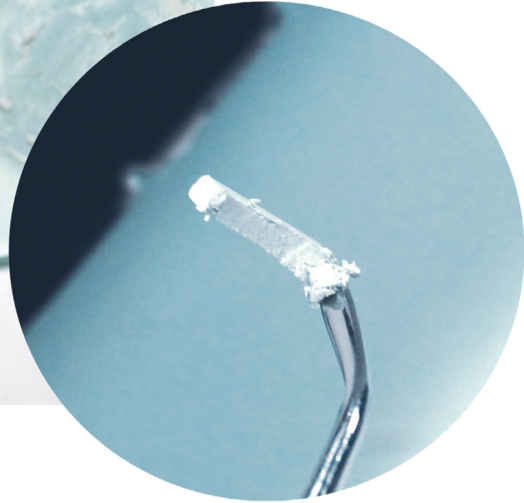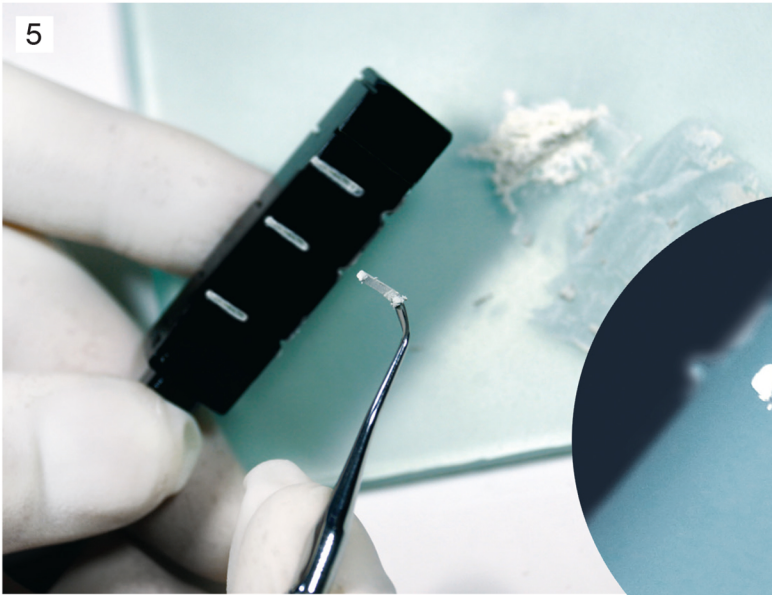

### WARNING

The product should be used within 10 minutes after preparing for application.  
Do not leave the product in the dosing block.

### PRECAUTIONS

Keep the instructions enclosed in the package leaflet of MTA+ product for correct future use.  
Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

### STORAGE

Keep out of the reach of children.

### HANDLING OF THE MATRIX

The packaged instruments are non – sterile.  
Before use: wipe with isopropyl or ethyl alcohol, leave to evaporate.

After use: remove the remains of MTA+ product using a brush and water with detergent.  
Clean the dosing block by wiping it with isopropyl or ethyl alcohol.

### Sterilization

Recommended sterilisation methods:

Method	Minimal parameters:
Steam autoclave	121°C, for 20 minutes



Last update: 17.01.2023/6



PPH CERKAMED Wojciech Pawłowski  
37-450 Stalowa Wola, Polska  
ul. Kwiatkowskiego 1  
Tel/fax 00 48 15 842 35 85

To maintain the high quality of our products and customer satisfaction, please send your opinions and comments to [jakosc@cerkamed.pl](mailto:jakosc@cerkamed.pl)